



3 Drops

**2 Results
Of Diseases**

Flu A/Flu B Antigen Rapid Test Kit

(immunochromatographic assay)

Influenza A virus

Influenza B virus

Intended Use

This kit is only used for the in vitro qualitative detection of Influenza A virus and Influenza B virus from human anterior nasal swab specimens. Flu A/Flu B Antigen Rapid Test Kit (immunochromatographic assay) is an immunochromatographic double-antibody sandwich assay intended for the qualitative detection and differentiation of Influenza A virus and Influenza B virus from individuals who are suspected of respiratory tract disease infection.

Advantages



Rapid

Results in 15-20 minutes .



Easy Process

Pre-filled extraction tubes simplify test procedure.



Multiple Samples:

Nasopharyngeal swabs, nasal swabs and oropharyngeal swabs.



Cost-effective Transportation

Room temperature storage and transportation .

Result Analysis


Influenza A		QIAstat-Dx Respiratory SARS-CoV-2 Panel		Total
		Positive	Negative	
Multiple Respiratory multipathogen antigen test kit (immunochromatographicassay)	Positive	91	1	92
	Negative	5	638	643
Total		96	639	735

Statistic	Value	95% CI
Sensitivity	94.79%	(88.26%~98.29%)
Specificity	99.84%	(99.13%~100.00%)
Total coincidence rate	99.18%	(98.23%~99.70%)


Influenza B		QIAstat-Dx Respiratory SARS-CoV-2 Panel		Total
		Positive	Negative	
Multiple Respiratory multipathogen antigen test kit (immunochromatographicassay)	Positive	87	1	88
	Negative	8	639	647
Total		95	640	735

Statistic	Value	95% CI
Sensitivity	91.58%	(84.08%~96.29%)
Specificity	99.84%	(99.13%~100.00%)
Total coincidence rate	98.78%	(97.69%~99.44%)


Operation Instructions

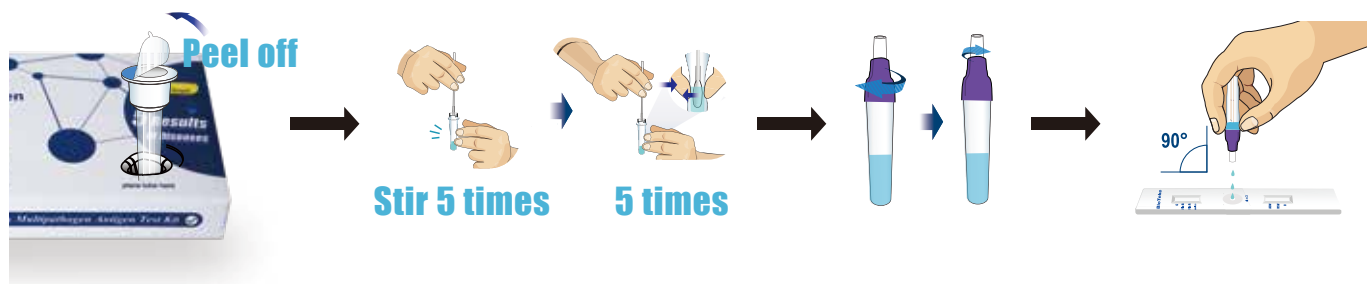


Result Interpretation

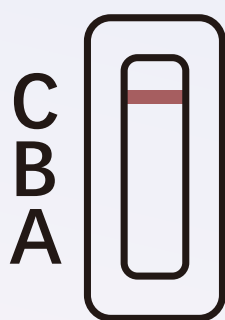


Negative

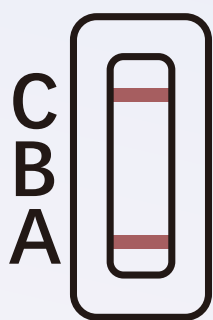

Flu A positive

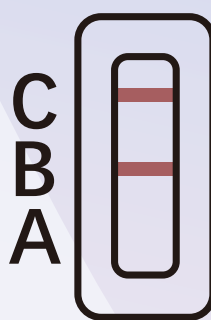

Flu B positive

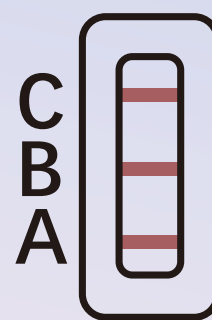

Flu A&B positive



Read the results within 15-20 minutes.

**BioTeke
Corporation**

BioTeke Corporation (Wuxi) Co., Ltd.

Version: 1.0

Revision date: Jan 06, 2023

For more information,
you can call BioTeke national service hotline:

0086-510-68501244

Or visit the official website
of BioTeke:

www.bioteke.cn

